



# **Owner's Manual**

## **Cleaning Instructions**

PRM Inc. cushions are finished using one of the following three options:

- Vacuum formed vinyl or Dartex can be wiped clean using mild soap and water. Do not submerge the cushion in water or autoclave the cushion, this will damage the internal foam. Do not use aggressive detergents, the fabric color may fade as a result.
- Vacuum formed neoprene can be wiped clean using mild soap and water. The top fabric layer will absorb some moisture but will not penetrate the entire cushion. Do not submerge the cushion in water or autoclave the cushion, this will damage the internal foam. Do not use aggressive detergents, the fabric color may fade as a result.
- Sewn Removable Dartex or Neoprene can be machine washed, in cold water on the delicate cycle. Hang to dry.

If you have any questions regarding your cushion contact the vendor who provided it.

## **Mounting to Wheelchair**

PRM Inc. does not attach the seating system to the wheelchair. Therefore, it is the responsibility of the vendor who provided your system to properly attach the seating to the wheelchair. Mounting hardware should be checked by the end user daily to ensure hardware is securely attached. Discontinue use if hardware has loosened and/or not properly attached. If you have any questions regarding your cushion contact the vendor who provided it.